

# Mestopore® S

## ABSORBENT CONTINENT STOMA DRESSING WITH SOFT SILICONE ADHESIVE


Mestopore S is a self-adhesive absorbent dressing with soft silicone adhesive for continent stomas. Mestopore S gives less trauma to the surrounding skin and less pain to the patient during dressing changes.



### Safetac® technology

#### Safetac® technology. Less trauma. Less pain.™

Safetac is a patented soft silicone adhesive technology. Dressings with Safetac technology are atraumatic both during wear and upon removal. These dressings minimize trauma to the wound and the surrounding skin, which minimizes pain to the patient. They also prevent maceration by sealing around the margin of the wound to protect peri-wound skin.

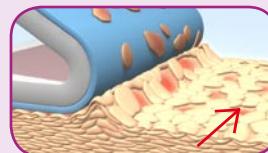

## How Mestopore S works


Mestopore S is a self-adherent, absorbent dressing with super-absorbent fibres and Safetac soft silicone micro-adhesive technology. A non-adherent contact layer protects the stoma tissue while the highly absorbent 'super-absorbent fibres' absorb and retain large amounts of fluid and secretions. The self-adherent border prevents leakage and is non-irritating, even with frequent application and removal. The outer film material is moisture vapour

permeable and water-proof for optimum comfort and security. Mestopore S can be lifted and re-applied to accommodate irrigation or inspection of stoma site.


## Areas of Use

- Continent Stomas
- Mucous fistula


## How to use Mestopore S

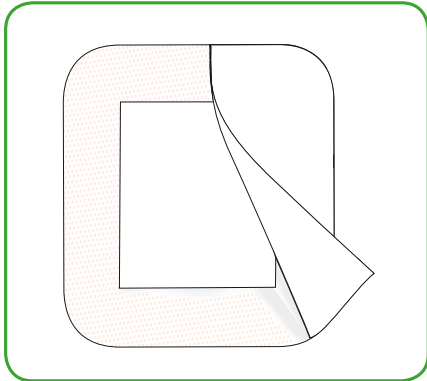

1. Remove 'release film' from bottom half of Mestopore S.

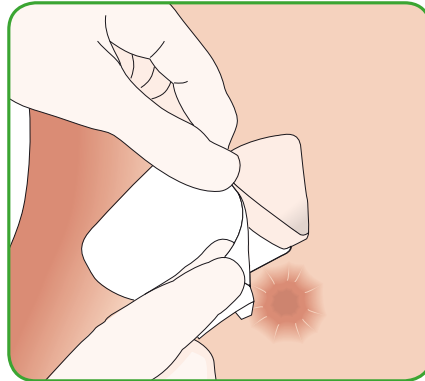

2. Apply to skin under stoma, then lift dressing upward over stoma. Remove 'release film' from top half of Mestopore S, press the dressing gently against skin above stoma.

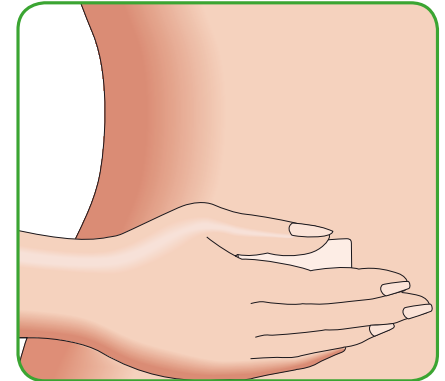

3. Gently smooth adhesive border against the skin to ensure secure fixation. To remove dressing, gently lift from "side-to-side" (not "top-to-bottom").

## Precautions

In case of clinical signs of infection, consult a health care professional for adequate infection treatment.

## Benefits of Mestopore S

- Minimizes pain and tissue damage during dressing changes
- Non-sensitizing
- Adapts well to body contours
- Shower-proof
- No residue
- Easy to use

## Mestopore S



## Note

Mestopore is also available with a traditional adhesive border.

- Water-based, non-sensitizing polyacrylate adhesive for gentle and secure fixation.

### Mestopore Assortment

Art. No	Size cm	Pad size cm	Pcs/box	Pcs/case
<b>Mestopore® S</b>				
665100	9 x 10	5 x 5.5	20	160
<b>Mestopore® with traditional adhesive</b>				
665000	9 x 10	5 x 5.5	20	160