


Mepore® Film


Breathable, transparent
film dressing

- Transparent, breathable, elastic polyurethane film for patient comfort and optimized wound healing environment

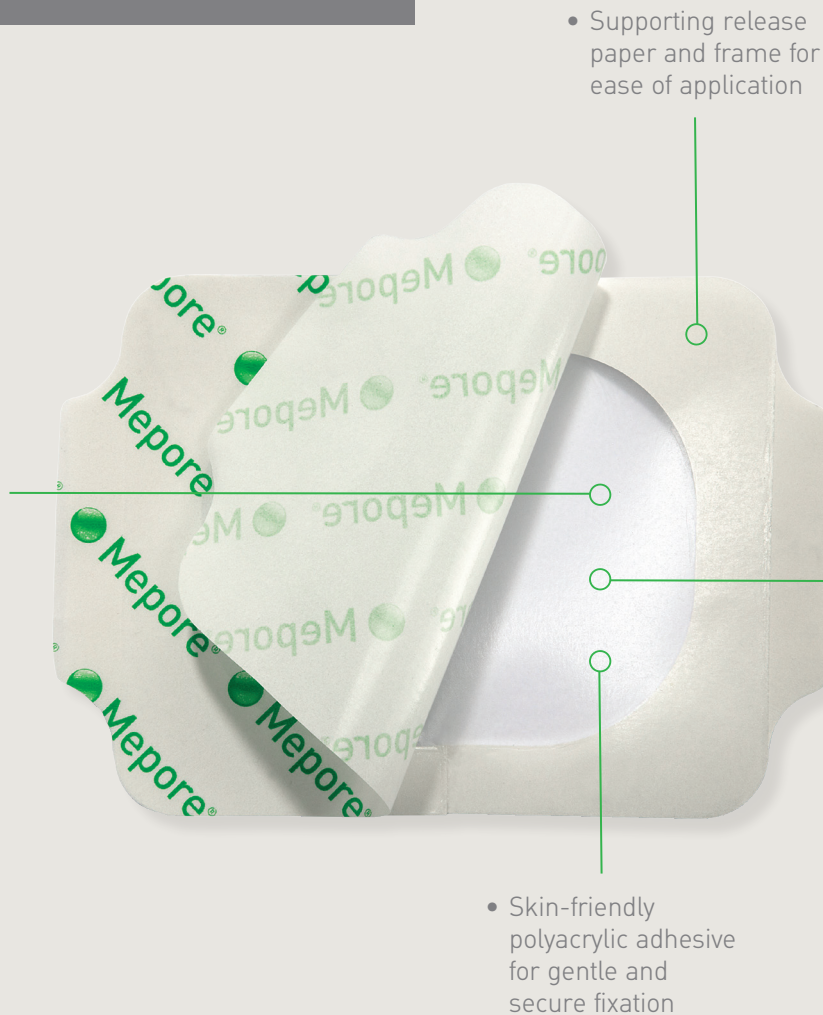


- Mepore® Film is a thin, transparent and moisture vapour permeable film dressing with a gentle, skin-friendly adhesive. It can be used as a primary dressing on clean, non-exuding wounds, abrasions and IV-sites, for prevention of skin breakdown or as a secondary dressing.

Directions for use

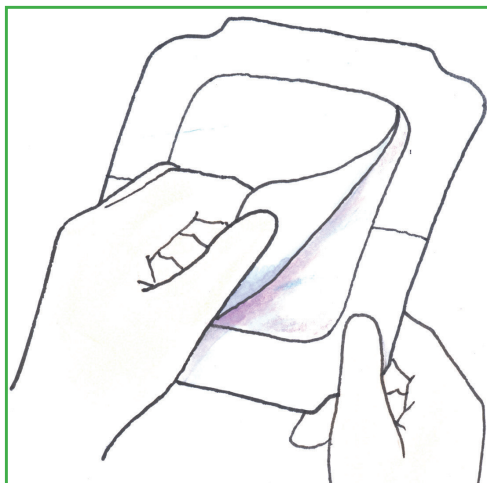


1. Choose a dressing size that is large enough to allow the dressing to adhere to dry, healthy skin around the site. The dressing may be trimmed or overlapped. Gently cleanse the wound area and dry surrounding skin.

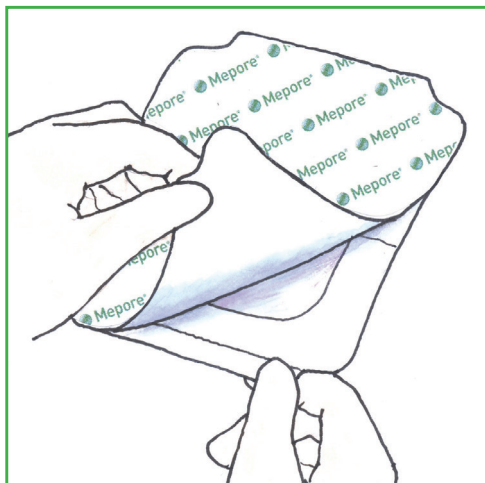

2. Remove imprinted backing paper to expose adhesive and apply dressing.

Note

To remove, lift dressing edge and pull gently in a horizontal direction parallel to skin surface.

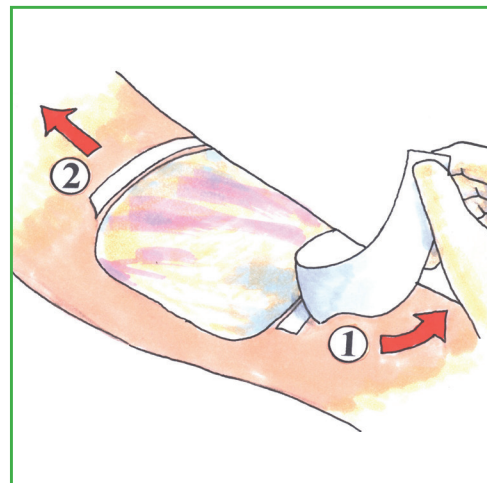

3. Remove white backing sheet (for sizes 10x25cm and 15x21.5cm) and "picture frame" tabs. Smooth edges down if required.

How Mepore® Film works

Mepore® Film provides a flexible, transparent covering to protect the wound, while conforming to body contours. The dressing provides a barrier to fluid strike-through and viral and bacterial contamination.

Mepore® Film maintains a moist environment in the wound. Due to its high vapour permeability, Mepore® Film allows excess moisture to pass away from the wound and surrounding skin.

Benefits of Mepore® Film

- High breathability
- Easy application, also when wearing gloves
- Limits the risk for damage of newly formed tissue
- Conforms easily with body contours
- Skin-friendly adhesive
- Secure fixation
- Maintains a moist wound environment
- Viral and bacteria proof, and provides a barrier against leakage
- Fluid impermeable

Mepore® Film Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
270600	6 x 7	100	600
271500	10 x 12.7	70	210
272500	10 x 25	20	100
273000	15 x 21.5	10	50

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Indications*

Mepore® Film can be used as a conformable fixation dressing to cover primary dressings, as e.g. hydrogels, alginates, and foam dressings.

Mepore® Film is also designed for a wide range of clean wounds in the granulation phase, such as:

- Superficial burns
- Superficial pressure ulcers
- IV sites
- Closed surgical wounds
- Abrasions
- Prevention of skin breakdown
- Lacerations

Precautions*

Mepore® Film should not be used on full-thickness wounds involving muscle, tendon, bone or on third degree burns.

Mepore® Film should not be used for patients who are sensitive to acrylic adhesive.

