


Mepore® Film & Pad

Breathable, transparent surgical dressing with absorbent wound pad

- Skin-friendly adhesive for gentle and secure fixation

- Highly absorbent wound pad with a low-adherent wound contact layer for enhanced patient comfort and longer wear time

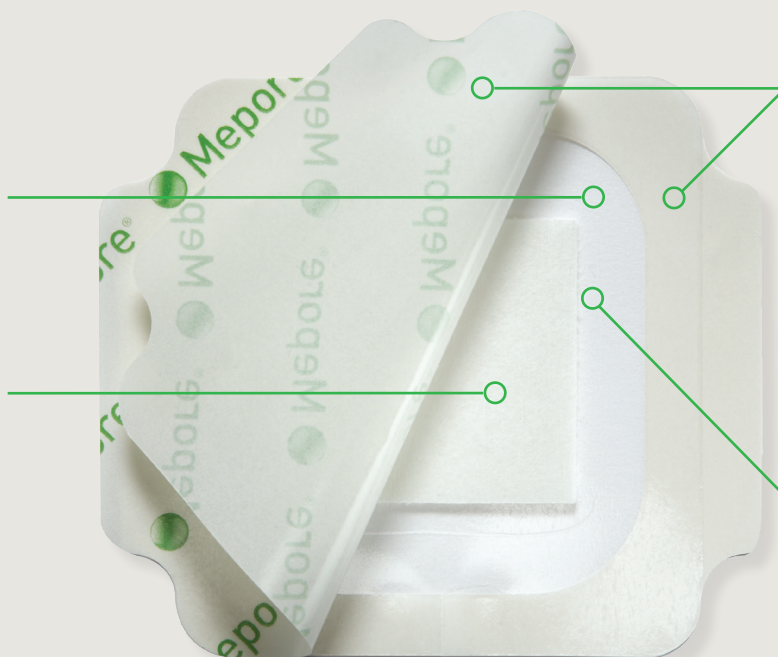

Supporting protection foil and paper frame for aseptic application

- Transparent, breathable, shower-proof film for patient comfort and optimized wound healing environment
- Viral and bacteria proof film, impermeable to fluids for safety and protection

- Mepore® Film & Pad is bacterial proof, viral proof and waterproof to protect the patient and the clinician. It is flexible and semi-permeable with a low adherent wound contact layer and a gentle, skin-friendly adhesive for optimal performance and comfort.

Mepitel® Film & Pad


Mölnlycke®

Directions for use



Gently cleanse the wound area and dry surrounding skin. Remove the release film and position the dressing over the wound to expose the wound pad and adhesive.



Smooth gently into place. Remove paper frames and the two white tabs.



To ensure proper adhesion, check that border edges are secure.

How Mepore® Film & Pad works

Mepore® Film & Pad incorporates a highly absorbent pad and high vapour permeable backing film to provide optimal handling. It provides a flexible, transparent and absorbent covering to protect the wound, while conforming to body contours. The skin-friendly polyacrylate adhesive is designed to be gentle to the skin while providing secure fixation.

Indications*

Mepore® Film & Pad is intended for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

Frequency of change

The frequency of dressing changes will be governed by the condition of the underlying wound.

Precautions*

- Mepore® Film & Pad should not be applied on patients who are sensitive to acrylic adhesive.
- Care should be taken to minimize skin tension that could cause mechanical damage to the skin.
- The wound should be inspected for signs of infection according to clinical practice. In case of clinical signs of infection (pain, redness, swelling or unusual odour or discharge) continued use of Mepore® Film & Pad should be determined by a health care professional.
- Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilize.
- If reused performance of the product may deteriorate, cross-contamination may occur.

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Benefits of Mepore® Film & Pad

- Low-adherent wound contact layer
- High breathability
- Excellent fluid handling
- Easy application when wearing gloves
- Conforms to body contours
- Viral and bacteria proof
- Gentle and secure fixation
- Waterproof
- Impermeable to fluids

Mepore® Film & Pad

Art. no	Size (cm)	Wound pad size (cm)	Pcs/Box	Pcs/Case
275100	4 x 5	1.5 x 2.5	85	340
275200	5 x 7, Oval	2.5 x 4	85	340
275300	5 x 7	2.5 x 4	85	340
275400	9 x 10	4.5 x 5	30	180
275500	9 x 15	4.5 x 10	30	240
275600	9 x 20	4.5 x 15	30	240
275700	9 x 25	4.5 x 20	30	150
275800	9 x 30	4.5 x 25	25	125
275900	9 x 35	4.5 x 30	25	100

