

# Mepore® Pro

Showerproof, self-adhesive and absorbent post-operative dressing

- Two part release film for easy application

- Skin-friendly, water based, solvent-free polyacrylate adhesive for gentle and secure fixation



- Semi-permeable transparent outer film supports a moist wound environment.
- Viral proof, bacterial proof and water proof.

- Absorbent wound pad with low adherent contact layer for enhanced patient comfort and longer wear time.

- Mepore® Pro is a self-adhesive, absorbent and semi-permeable dressing with an outer transparent film layer to protect the wound from water and contamination. Its skin-friendly, water-based, solvent-free polyacrylate adhesive provides secure yet gentle fixation. It is bacterial proof, viral proof and waterproof to protect the patient and the clinician.

Mepore® Pro

  
Mölnlycke®

## Directions for use



1. Remove from package and remove outer half of release film.



2. Gently fixate to skin at one end of incision site. Position dressing over incision site. DO NOT STRETCH.



3. Remove the remaining half of the release liner and smooth adhesive border to the skin.

## Frequency of change

Change Mepore® Pro as indicated by wound condition or existing protocol. To remove, loosen one edge and slowly peel the dressing from the skin in the direction of hair growth.

## How Mepore® Pro works

The transparent outer film enables the patient to shower while protecting the wound from water and contamination. It also protects clothing and bedlinen from blood or exudate strike through. It's highly absorbent pad, in conjunction with the high vapour permeability of the outer film, provides optimal fluid handling, promotes healthy peri-wound skin and is comfortable to wear. The stability of the fabric/film combination provides for easy application while the water-based polyacrylate adhesive provides secure "skin-friendly" fixation. Since Mepore® Pro is radiopaque, it can stay in place during x-ray.

## Benefits of Mepore® Pro

- Showerproof and radiopaque
- Low-adherent wound pad keeps wounds clean, minimizes the risk of adherence to the wound and reduces dressing change frequency
- Viral proof, bacterial proof and waterproof to protect the patient and the clinician
- Flexible, conformable to body contours, and easy to apply
- Skin-friendly, water-based adhesive
- Rounded corners for secure fixation (all sizes up to 9x15 cm)

## Mepore® Pro Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
670820	6 x 7	60	480
670920	9 x 10	40	320
671020	9 x 15	40	320
671120	9 x 20	30	180
671220	9 x 25	30	210
671320	9 x 30	30	270

\*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to [www.molnlycke.ca](http://www.molnlycke.ca)

## Indications\*

Mepore Pro is intended for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

## Precautions\*

- Mepore® Pro should not be applied on patients who are sensitive to acrylic adhesive.
- Care should be taken to minimize skin tension that could cause mechanical damage to the skin:
  1. Ensure that Mepore® Pro is not applied under tension.
  2. When applying over joints be careful to apply in such a way that it does not impair flexibility of joints.
  3. After some surgical interventions excessive swelling may occur. In such a case it could be better to use a more flexible dressing, such as Mepore® Film polyurethane film, in order to minimize skin tension.
- The wound should be inspected for signs of infection according to clinical practice.
- In case of clinical signs of infection (pain, redness, swelling or unusual odour or discharge) continued use of Mepore® Pro should be determined by a health care professional.

