



Mesorb®

Ultra absorbent dressing with exudate strike-through barrier

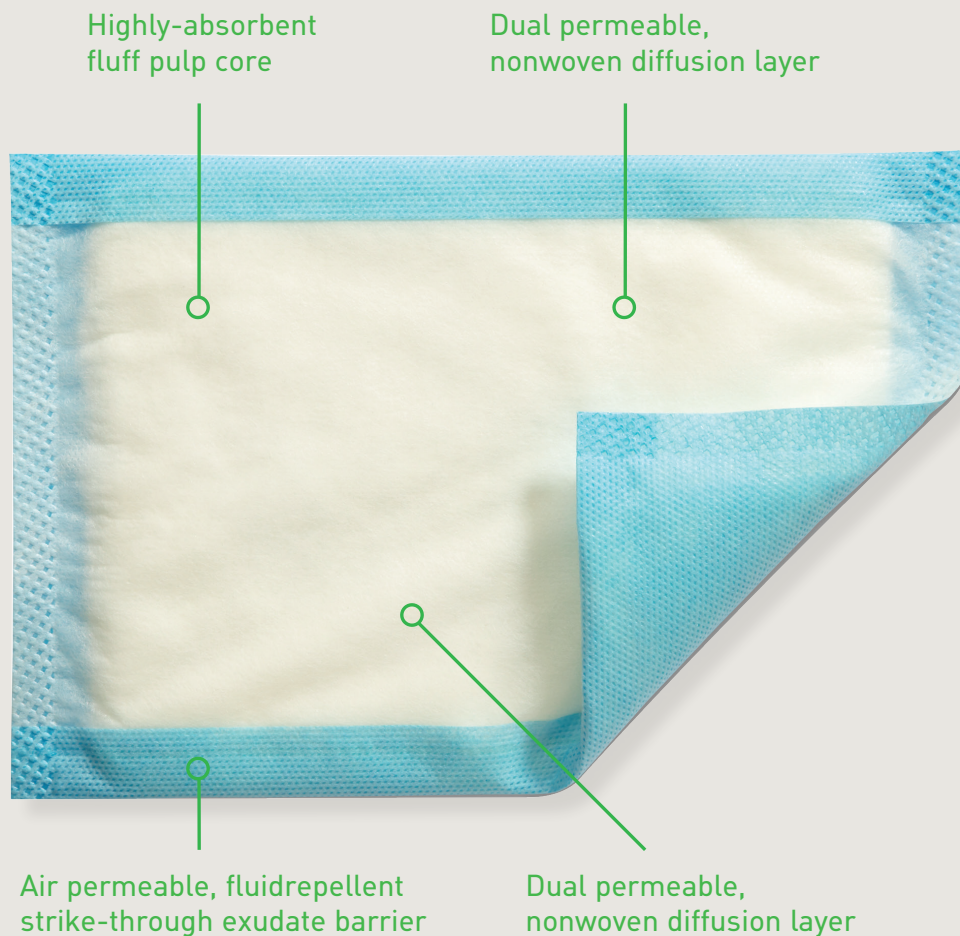


- Mesorb® is a soft highly absorbent dressing that can retain large amounts of exudate. The unique fluid repellent barrier reduces frequency of dressing changes and prevents soiling of clothing and bed linen.


Directions for use

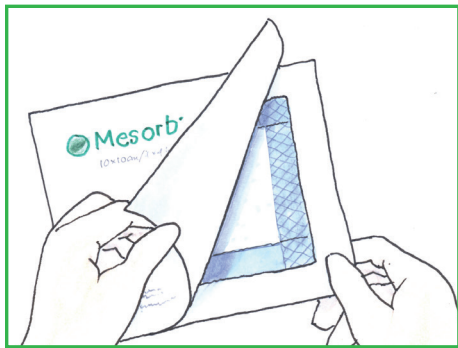

1. Select an appropriate size to overlap the wound area by 2 to 4 cm. Open the package and remove the dressing.

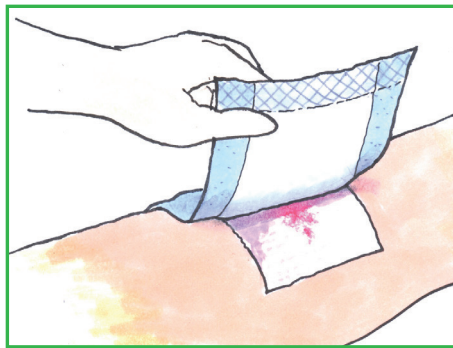

2. Lay Mesorb® over primary dressing or the wound area.

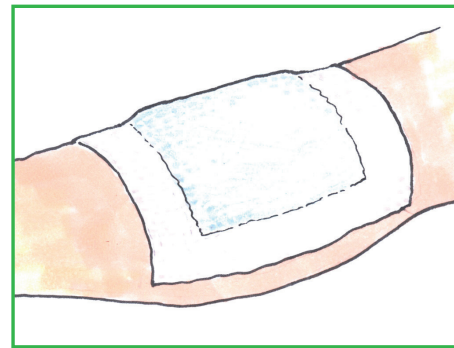

3. Fixate with Mepitac® soft silicone tape, Mefix® fabric tape or non-adhesive fixation product.

How Mesorb® works

Wound exudate is absorbed and transferred into Mesorb® by the wound contact layer and the diffusion layer. The highly absorbent core absorbs exudate, while simultaneously providing a good protective and ventilating wound cushion.

The fluid-repellent backing prevents external contamination of the wound and protects clothes and bed linen from exudate strike-through. As the backing covers the sides of the dressing, it also prevents side leakage.

Benefits of Mesorb®

- Highly absorbent
- Sealed on all sides
- Breathable
- Cost effective
- The unique strike-through exudate barrier reduces frequency of dressing changes and prevents soiling of clothing and bed linen

Indications*

Mesorb® is intended for use in moderate to heavily exuding wounds.

Mesorb® can be used as a primary dressing (directly on the wound), or as a secondary dressing where high absorbency is required.

Frequency of change

Mesorb® may be left in place for several days, depending on the wound and skin condition, or as indicated by a healthcare professional.

Precautions*

- Mesorb® should not be used on patients with a known sensitivity to the dressing or its components.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilize.
- Do not use after expiry date. If the product is used after the expiry date product properties cannot be ensured.

Mesorb® Assortment (Sterile packed)

Art. no	Dressing Size cm	Wound Pad size cm	Pcs/Box	Pcs/Case
677001	10 x 10	10 x 13	50	400
677201	10 x 15	13 x 15	50	300
677401	10 x 20	10 x 23	50	300
677601	15 x 20	15 x 23	50	300
677701	20 x 25	23 x 25	30	120
677801	30 x 20	23 x 30	30	120

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

