Mepilex® Border Post-Op

Applikasjonsguide

Mepilex® Border Post-Op. Bandasjen for kompromissløs behandling av kirurgiske sår.

Ivaretar pasientmobiliteten¹⁻⁴

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REFERANSER: **1.** Johansson C. et al. An assessment of a self-adherent, soft silicone dressing in post operative wound care following hip and knee arthroplasty. Poster presentation at EWMA, Brussels, Belgium 2012. **2.** Van Overschelde, P. et al. A randomised controlled trial comparing two wound dressings used after elective hip and knee arthroplasty. Poster presentation at 5th Congress of the WUWHS, Florence, Italy, 2016. **3.** Bredow, J et al. Randomized clinical trial to evaluate performance of flexible self-adherent absorbent dressing coated with silicone layer after hip, knee or spinal surgery in comparison to standard wound dressing. Poster presentation at 5th Congress of the WUWHS, Florence, Italy, 2016. **4.** Zarghooni, K. et al. Effect of a modern dressing compared to standard dressings on outcome after primary hip and knee arthroplasty: a prospective, non-randomised controlled study. E-poster presentation at EWMA conference, London, United Kingdom, 2015. **5.** Meaume S. et al. A study to compare a new self-adherent soft silicone dressing with self-adherent polymer dressing in stage II pressure ulcers. *Ostomy Wound Management*, 2003. **6.** Feili F et al. Retention capacity. Poster presentation at the EWMA conference, Lisbon, Portugal 2008. **7.** Wiberg A.B. et al. Preventing maceration with a soft silicone dressing: in-vitro evaluations. Poster presented at the 3rd Congress of the WUWHS, Toronto, Canada, 2008. **8.** Johnstone, A. et al. Innovations in the reduction of pressure ulceration and pain in critical care. *Wounds UK*, 2013. **9.** Feili, F. et al. A laboratory evaluation of the fluid retention properties of post-operative absorbent dressings. Poster presented at the 5th Congress of the WUWHS, Italy, 2016. **10.** Feili, F. et al. Fluid handling properties of post-operative wound dressings. Poster presentation at 5th Congress of the WUWHS, Florence, Italy, 2016. **11.** Feili F. et al. Blood absorption capacity of post-operative wound dressings. Poster presented at the 5th Congress of the WUWHS, Italy, 2016. **12.** Johnstone, A. et al. Innovations in the reduction of pressure ulceration and pain in critical care. *Wounds UK*, 2013.

Les mer på www.molnlycke.no

Mölnlycke Health Care AS, Postboks 6229 Etterstad, 0603 Oslo. Tlf. 22 70 63 70
The Mölnlycke, Mepilex and Safetac trademarks, names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. © 2022 Mölnlycke Health Care AB. All rights reserved. NOWC0312209

Mölnlycke®