# Handleiding

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**Referenties:** 1. Nelson, D. A New Bordered Foam Dressing Technology Improves Wound Outcomes and Satisfaction While Reducing Dressing Utilization in Acute Care. Poster Presentation at WOCN Annual Meeting, Nashville, 2019. 2. Tyson, L. Study First: Driving the Case for Improving Hospital Wound Care. Poster Presentation at WOCN Annual Meeting, Nashville, 2019. 3. Mölnlycke Health Care. Mepilex® Border Flex – Estimate of spreading area using dot patterns on backing film. Report no. PD-528872. Data on file. 4. Mölnlycke Health Care. Mepilex® Border Flex Product Manual – Conformability PD-528870. Data on file. 5. Mölnlycke Health Care. Mepilex® Border Flex-Bacteria encapsulation. Report no. PD-537072. Data on file. 6. Mölnlycke Health Care. Mepilex® Border Flex Product Manual – Spreading layer PD-528874. Data on file. 7. Mölnlycke Health Care. Product Manual – Maintained dressing profile. Report no. 525458. Data on file. 8. White, R. A multinational survey of the assessment of pain when removing dressings. Wounds UK 2008;4(1):14-22. 9. Mölnlycke Health Care. Mepilex® Border Flex – Absorption and dispersion tests on inclined plane, viscous test solution. Report no. PD-528871. Data on file. 10. Mölnlycke Health Care. Product manual Initial abs different test solutions. Report no. PD-535352. Data on file. 11. Mölnlycke Health Care. Mepilex® Border Flex – Fluid handling capacity compared to Urgotul® Foam Absorb Border. Aquacel Foam, Allevyn Classic Gentle Border, Allevyn Life, Opflteam Gentle, Kerraf Foam Gentle Border. Report no. PD-527642. Data on file. 12. Mölnlycke Health Care. Product Manual Mepilex® Border Flex – Viral penetration. Report no. PD-535090. Data on file. 13. Opinion Matters. Professional Health Care Treatment for Wounds survey conducted June-July 2018 in UK. Data on file. 14. Woo, K. & Bergström, C. An Open Randomised Cross-over Investigation Assessing Perceived Pain at Dressing Change Comparing A Soft Silicone Dressing, Mepilex® Border, with a Adhesive hydrocolloid polyurethane dressing, Allevyn® Adhesive in Patients with chronic ulcer (Clinical Investigation Report IMX8468, Part A, Mölnlycke Health Care). 2007. 15. Van De Looerbosch, D., Andersson, A., Husmark, A. & Körner, V. An open randomised explorative study comparing Mepilex® Border with Tielle in patients with pressure ulcers according to stage II, EPUAP guidelines (Clinical Investigation Report IMPX8201), Mölnlycke Health Care, 2001. 16. Mölnlycke Health Care. Mepilex® Border Flex – Waterproofness. Report no. PD-532095. Data on file.

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